

QUICK REFERENCE

CORRIDOR4DM FEATURE KEY UPDATE GUIDE

Corridor4DM Version:
6.0 and 6.1

Target Audience:
IT Personnel
Physicians

Primary Workstation:
Stand-alone PC or
Laptop

NOTE:
HASP = Hardware Against Software Piracy. The Dongle (HASP security USB stick) enables the user to launch 4DM. If it is a Local (purple) HASP, then it must be plugged into the local machine. If it is a NET (red) HASP, then it can be plugged into any machine on the local network.

Note: If an error occurs during the update or you need assistance, please call our Product Support line at (734) 205-1231, x501.

To update your Corridor4DM Feature Key information, follow these step-by-step instructions:

- Insert the Dongle (HASP security USB stick) into the computer.
- Launch Corridor4DM.
- Click **Preferences** on the **Application Control Panel** (circled in blue in Figure 5).
- Under **Global Settings**, click **Feature Keys** (circled in pink in Figure 6).
- In the **License Key** field (circled in green in Figure 6) type in your new 18-digit **License Key** *exactly* as it appears on the Welcome Letter from INVIA.
- Click the **Check Status** button.
- The **Status** field will confirm the new license key and provide a new expiration date in year-month-day format (Figure 7).
- Click **Save**. This will automatically close the Corridor4DM **Preferences** window.

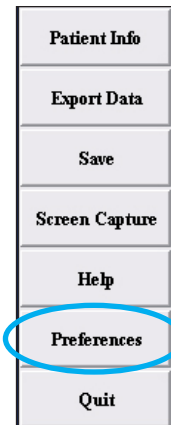


Figure 5: Application Control Panel

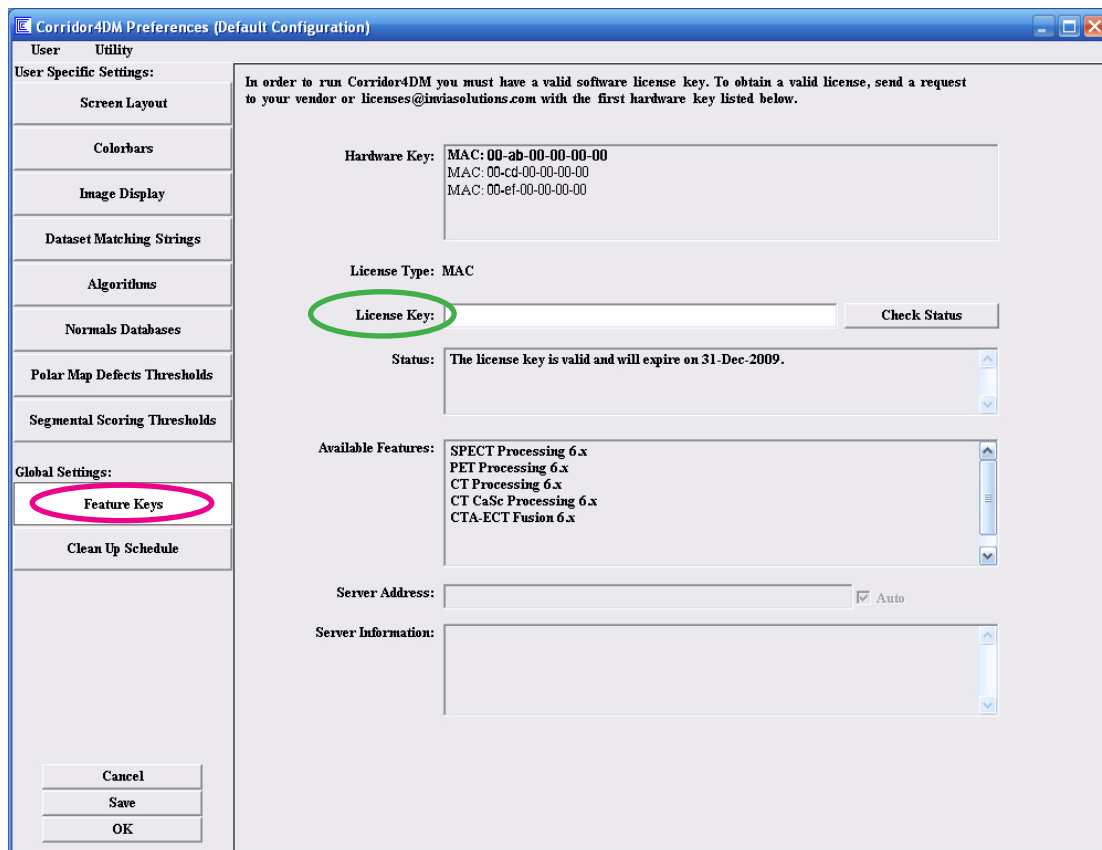


Figure 6: Shows the **Feature Keys** screen within the **Preferences** menu. The **Feature Keys** button is circled in pink. Input the 18-digit software key in the **License Key** section, circled in green.



Figure 7: Shows the **Status** section once a valid software key has been entered in the **License Key** section.